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Preservation methods: Store the product at room temperature, protected from light and humidity, in ventilated environments, raised off the floor, in its own protective packaging. Keep away from excessive temperatures or humidity. **Storage:** Irradiation. **Disposal:** Dispose of the DM after use in accordance with the national laws in force for potentially contaminated DM.

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ISTRUCTIONS FOR USE

ENGLISH

Brand Name: DERMADINE PLUS incision film with iodine antiseptic Dermadine Plus is in class III according to the directive 93/42/CEE

Composition: DERMADINE PLUS: Polyurethaneplastic resin, acrylic adhesive iodine potassium iodide

⚠️ⓘ Read these instructions and warnings carefully before using Medical Device (MD).

Indications of use: Dermadine Plus are intended to protect the skin in surgical procedures, acting as a mechanical barrier which prevents the transfer of microorganisms before incision. Avec Dermadine Plus l'antiseptique auxiliaire action de iodine supports the disinfection and barrier activity exerted by the drape during the intervention.

Special warnings: MD is a non-conductive plastic material, so for defibrillation or procedures requiring the use of electrical devices, remove the film from the skin.

Warnings on adhesive capacity: To guarantee maximum adhesion ensure that the skin is perfectly dry before application. Make sure no creases or air bubbles form when applying the film. **Warnings on the adhesive:** The appearance of dark brown or amber colored dots or marks on the MD Dermadine Plus adhesive is not an imperfection or alteration of the product. **Contraindications:** Do not use on patients with acknowledged sensitivity to materials used to make Dermadine Plus. **Iodine side effects:** do not use Dermadine Plus in people with known sensitivity to iodine or iodophor compounds. The interaction of iodine with acetone can lead to the formation of irritating compounds. **Dermadine Plus** is biocompatible but effects such as burning, iododerma, acne-like eruptions, decrease in the rate of scarring of the tissues, irritation of the mucous membranes may occur at the level of the skin. In the case of allergic reaction, remove the towel, contact the doctor and notify the hospital pharmacy. **Conseils d'utilisation:** Fig. 1 Retirer la face li inciser de l'emballage stérile. Enlever la protection additionnelle en papier.

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Fig. 4 With the film held taut apply a few centimeters from the patient's perfectly dry skin, apply it to the surgical area according to requirements.

Fig. 5 Use a sterile gauge, to apply pressure along the incision line and smooth outwards without stretching the film excessively. Make sure there are no creases or air bubbles, which could compromise the film's adhesion properties, during surgery.

Fig. 6 To remove the film, peel it off the skin gently at an angle of 180 degrees and hold it only with the fingers. Avoid excessive pulling. Take extreme care when removing the film from the skin with little elasticity so as not to injure or damage the patient's skin.

Precautions and Warnings: Disposable. The sterility of MD is guaranteed when the pack-aging is sealed. Use for sterile procedures only. Discard all damaged or unsealed packages. MD must be used by qualified operators only. The user is responsible for using the product correctly. MD is for external use only. MD is designed for single use on a single patient. Re-sterilization or reutilization is prohibited. Do not use the MD after the expiry date indicated on the package.

Preservation methods: Store the product at room temperature, protected from light and humidity, in ventilated environments, raised off the floor, in its own protective packaging. Keep away from excessive temperatures or humidity. **Sterilising agent:** Irradiation. **Disposal:** Dispose of the DM after use in accordance with the national laws in force for potentially contaminated MD.

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TIASET, DERMADINE PLUS is TIASET of Carresi Lorenzo registered trademarks.